



# Share Your Blood Glucose Data With Your Healthcare Professional (HCP)

Sharing your readings helps your HCP support your diabetes management, even between appointments.

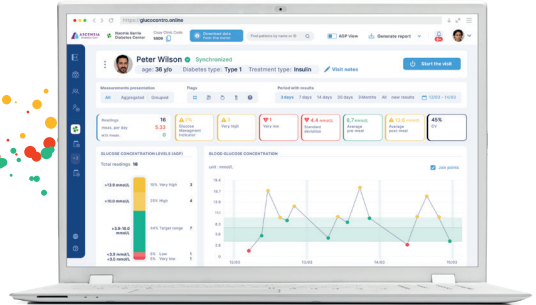

**Contour  
next**  
Blood Glucose Monitoring System

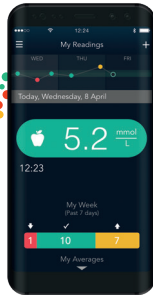

**Contour**  
diabetes app



**GlucoContro**  
.online

## Benefits:

- ✓ **Automatic syncing** from your CONTOUR®NEXT meter and CONTOUR®DIABETES app
- ✓ **No extra steps after setup** - data sharing runs in the background
- ✓ **Seamless access for your HCP** to support your care remotely



Scan to learn  
about the Connected CONTOUR® Solution

Get the free CONTOUR®DIABETES app here:



**Contour**  
diabetes app



Check to see if your smartphone's operating system is compatible\*\*



# Link Your Account With Your HCP

A quick and easy way to remotely share your blood glucose readings.

**Before you begin: Set up your CONTOUR®DIABETES app account**

These login details also need to be used for GlucoContro.online (GCO) to ensure your data syncs correctly.

Email: \_\_\_\_\_

Choose one of the options below.



## Email Invitation

1. Click **Receive the Invitation**, sent to you by email
2. **Sign in**, or **Create your Account** using the same login you use for the CONTOUR®DIABETES app. Select **Sign Up**
3. **Verify your account** via the **email link**, follow the prompts, review the Privacy Policy, and click **Continue**
4. **Login** to GCO and complete your profile
5. When prompted, **Accept** your HCP's invitation and finalise your profile settings

HCP Clinic Code:

## Clinic Code or QR code

1. Go to **GlucoContro.online** and **login**
2. From the menu, select **Enrol in Clinic**
3. Enter the **clinic code** your HCP gives you or **scan their QR code**
4. Your clinic will approve the request

**TIP**

You only need to set GCO up once. After that, your data will upload from your CONTOUR®DIABETES app automatically.

Visit [www.diabetes.ascensia.com.au](http://www.diabetes.ascensia.com.au)

Always read instructions for use. Consult your healthcare professional if this product is right for you.

\*\*On a compatible Android or iOS device. For a list of compatibility devices, please visit [compatibility.contourone.com](http://compatibility.contourone.com)  
SdNcenter is the legal manufacturer and developer of GlucoContro.online and is partnering with Ascensia to market and deliver the tool to HCP and Patient users.

© Copyright 2026. Ascensia Diabetes Care Holdings AG. All rights reserved. Ascensia, the Ascensia Diabetes Care logo and Contour are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.