

Access shared data remotely and in real time*


To assist with blood glucose management, invite your clients with diabetes to share their data.

www.glucocontro.online

GlucoContro.online

Easy, centralised access to your clients' data

Available on one screen with a single click

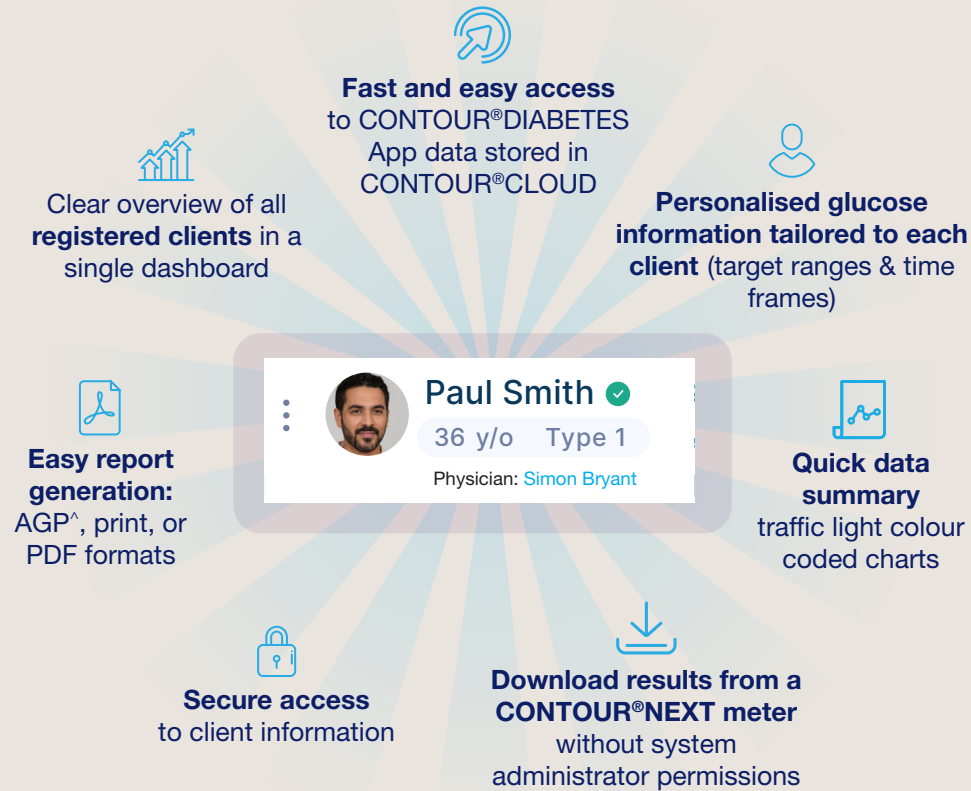

Free download




For a list of compatible mobile operating systems, visit compatibility.contourone.com

[^]Ambulatory glucose profile

A comprehensive view for blood glucose readings

To provide understanding of glucose values

Quick access menu

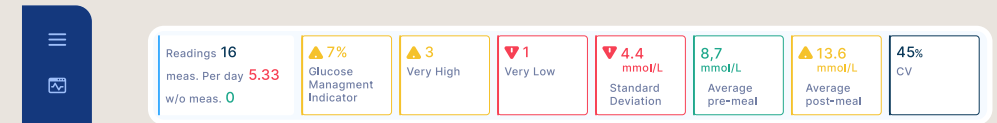


Summary: Key insights to support clinical decision making



Percentage breakdown of readings by glucose range



Distribution of readings: View glucose variability at a glance

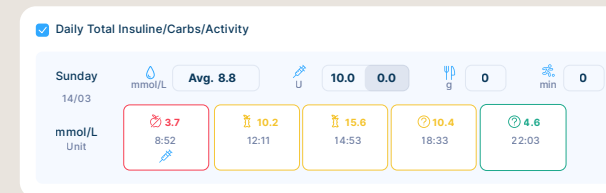

Measurements table: Simple display of all collected data for each day

	Total count	Average	Very High	Very Low	High	Target	Low	Std. dev.
Fasting	16	9.5 mmol/L	8%	3%	50%	38%	13%	4.3 mmol/L
Pre-meal	3	3.8 mmol/L	0%	33%	0%	33%	67%	1.6 mmol/L
Post-meal	3	8.7 mmol/L	0%	0%	67%	33%	0%	2.4 mmol/L
No flag	5	13.6 mmol/L	60%	0%	80%	20%	0%	3.4 mmol/L
All	5	9.3 mmol/L	0%	0%	40%	60%	0%	2.9 mmol/L

Results summary: A detailed overview of client blood glucose data

GlucoContro.online

Set up a new healthcare professional (HCP) account and invite a client to share their data

Register in 3 simple steps

1. Visit **GlucoContro.online** and click **Sign up**
2. Complete the Professional Account Registration form
3. Wait for the approval from the clinic administrator†

Click “Add new PWD”

Create an account

Account type

Patient
Choose this type of an account if you have been diagnosed with diabetes.


Professional
Choose this type of an account if you are a healthcare professional.

Care Giver
Choose this type of an account if you want to take care of someone with diabetes but you are not a diabetic.

E-mail address
Enter e-mail address

First name
Enter first name

Last name
Enter last name

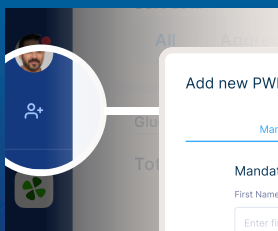

Add new PWD

Manual QR Code

Mandatory

First Name
Enter first name


Last Name
Enter last name

Country
Australia

Optional

I want to send an e-mail invitation to this patient

Add new Patient

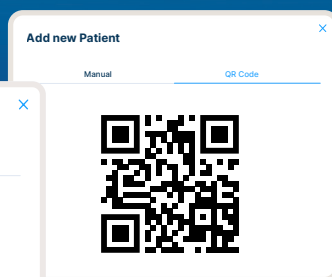

The indicates that the client has accepted the invitation

Paul Smith

36 y/o Type 1

Physician: Simon Bryant

Sign up at **GlucoContro.online**

When it comes to blood glucose management, **trust CONTOUR®**

diabetes.ascensia.com.au | 1800 289 312 @ContourAscensia.AU

For healthcare professionals only.

*People with diabetes need to be registered in GlucoContro.online and approve data sharing with their healthcare professional. Data available as soon it is uploaded to the CONTOUR®CLOUD.

† It may take up to 24 hours

SdNcenter is the legal manufacturer and developer of GlucoContro.online and is partnering with Ascensia to market and deliver the tool to HCP and people with diabetes users.

© 2026. Ascensia Diabetes Care Holdings AG. All rights reserved. Ascensia, the Ascensia Diabetes Care logo and Contour are registered trademarks of Ascensia Diabetes Care Holdings AG.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC All other trademarks are properties of their respective owners and are used solely for informative purposes.

