

Microlet™ next Lancing Device



Steps to lancing success



Ascensia Diabetes Care Holdings AG,
Peter Merian-Strasse 90, 4052 Basel, Switzerland.

Ascensia, the Ascensia Diabetes Care logo and Contour are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

© Copyright 2018 Ascensia Diabetes Care Holdings AG. All rights reserved.



1 Prepare



Twist and unlock the endcap as shown

Line up the white raised mark and black indented mark



Loosen the round protective cap on the lancet but do not remove it

Rotate 1/4 turn



Insert the lancet firmly into the lancing device

Push until the lancet comes to a complete stop



Carefully twist off the round protective cap

Save the cap for disposing of the used lancet



Re-attach the endcap

'Click' to lock

2 Lance



Rotate the endcap dial to adjust the puncture depth

1 for the shallowest and 5 for the deepest



Pull back the reset handle until the release button is raised



Press the lancing device firmly against the side of your finger and press the release button

You will hear a click when the lancet is released

3 Eject



Remove the endcap

Place the protective cap that you saved onto a flat surface. Push the lancet into the cap as shown



Push the grey lancet ejector forward until the lancet falls into the container, then slide ejector back

Don't forget to re-attach the endcap

Before use and for full information please see the MICROLET™NEXT packaging insert¹